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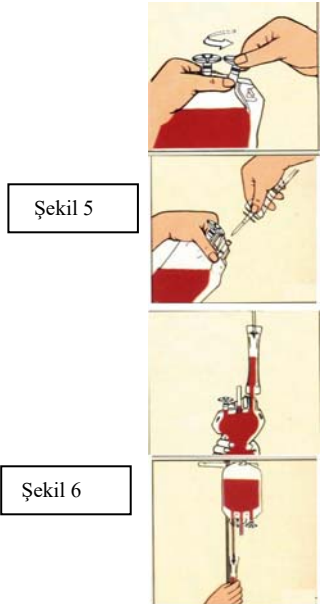
**DESCRIPTION**

Transfer bag system made of PVC, including transfer spike, non-vented cap and a blood bag.

A.22000006515

B. 22004006500

C. 22001006504

D. 22004006505

Figure 1

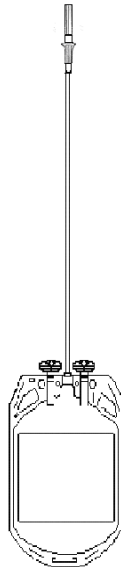
Figure 2

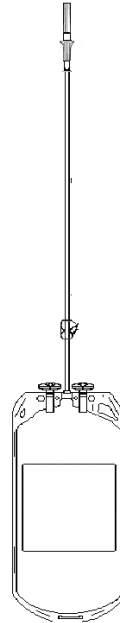
Figure 3

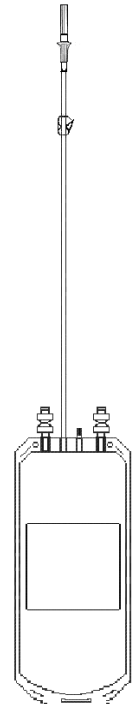
Figure 4

**⚠ PRECAUTIONS**

1. This product is intended to be used by certified Blood Bank personnel in the Blood Bank.
2. Do not use if spike protector is opened.
3. Cross match the contents of the bag with recipient before transfusion.
4. Use transfusion sets in compliance with EN ISO 1135-4.
5. Follow validated blood component preparation procedures according to local requirements.
6. The tubes are suitable to be connected with a Sterile Connecting Device.
7. It is intended for human blood and components.
8. During shipment, up to two shipping boxes may be carried at the same time manually or on a trolley, they must not be dropped or any heavy items loaded on top of them and they must be loaded in such a way to prevent them from falling over.
9. Single use only, after usage it should be thrown as a Medical Waste. **Caution:** There is a risk of infection if used more than once.
10. For the explanation of symbols, see Symbols Description Table.

**Caution:** This medical device contains di(2-ethylhexyl) phthalate (DEHP). Children including developing foetus, premature neonates and neonates and therefore potentially pregnant and nursing women are at higher risk to DEHP exposure, although humans have been observed to be less sensitive than rodents. Furthermore, exchange transfusion of blood in neonates, extracorporeal membrane oxygenation treatment of neonates and adults, total parenteral nutrition in neonates, haemodialysis, enteral nutrition in neonates and adults, heart transplantation or coronary artery bypass graft surgery, massive blood transfusion of red blood cells and plasma or peritoneal dialysis may lead to high exposure to DEHP. Medical procedures should not be avoided however, as the benefits outweigh any possible health risks associated with DEHP exposure. Please refer to current literature to make an informed decision.

**INDICATIONS**

For storage, processing, transportation and transfusion of blood and blood components per “*Guide to the Preparation, Use and Quality Assurance of Blood Components*”.

**GENERAL PREPARATIONS BEFORE USAGE**

1. Before opening transfer bag systems, check the overwrap, tears, punctures, etc.
2. After opening the overwrap, check whether the transfer bag is torn and punctured.
3. Do not use defective sets. **Caution:** In products gamma irradiated, the set may give off a different smell because of gamma irradiation.

**INSTRUCTIONS FOR USE**

1. Remove the system from its outer package.

2. Connect the system to another bag;

- a) Manual Connection;
  - i) Remove the transfer spike's protective cap above the transfer tube.
  - ii) Insert the spike firmly into the outlet port of the blood bag.
- b) Connection with Sterile Connecting Device;
  - i) Connect the transfer tube to the tube of bag that blood component will be transferred with a Sterile Connecting Device. **Caution:** Please see Sterile Connecting Device's user instructions.

3. Blood component separation is performed by manual or automatic extractor according to the manufacturer's user instructions.

4. After separating the blood components, seal the tubes with the Sterile Connecting Device from the closest point of the transfer bag according to the manufacturer's user instructions.

#### **STORAGE**

1. If whole blood and/or RBCs are stored, they should be placed into blood bank refrigerators designed for this process under 1-6 °C.
2. If platelets are stored, pooled platelets can be stored at 22 °C up to 6-8 hours in an agitated incubator.
3. If plasma is stored, store the plasma in the freezer. If the separation takes place within 6 hours after blood collection, it can be stored at -18 °C/-25 °C for 3 months, -30°C/-40°C for 6 months and at lower temperatures for 12 months.

**Caution:** While removing blood bags from the blood bank refrigerator in case of need, it is very important to handle them carefully so that they are not exposed to hard and sharp objects that could damage them and not rubbed to the floor and dropped or thrown from a high distance while placing them on the table.

**Caution:** After removing the blood bags from the blood bank refrigerator, they should be kept at room temperature for 4-10 minutes to return to the normal temperature since the PVC structure of the blood bag has a fragile nature at colder temperatures.


#### **TRANSFUSION**

1. Inspect the container and blood for any defects.
2. Before transfusion, make sure that serological test results are acceptable and cross match test results with the recipient. If appropriate, perform the cross match test again at bed side.
3. Platelets should be inspected before transfusion to ensure that no platelet aggregates are visible.
4. Use transfusion set compatible with EN ISO 1135-4.
5. Hold the top part of the T part with one hand and bag from the bottom part of T part with other hand, and twist of the upper part by 270° (Figure 5).
6. Keep holding the bag from T part, remove the cap of the transfusion spike with other hand.
7. Insert the spike by ¾ of its length into outlet port. Do not touch the spike of the transfusion set and open outlet port to prevent contamination.
8. After inserting the spike, squeeze the bag to the level that transfusion set filter is filled with blood (Figure 6).
9. The unit is ready for transfusion. Hang the bag onto the stand (Figure 6).

Figure 5

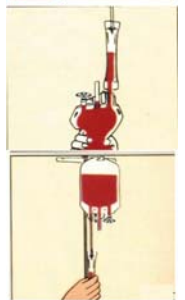Figure 6

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